



DEKO 190GT WASHER DISINFECTOR - LOADING INSTRUCTIONS

(version: 08/20)

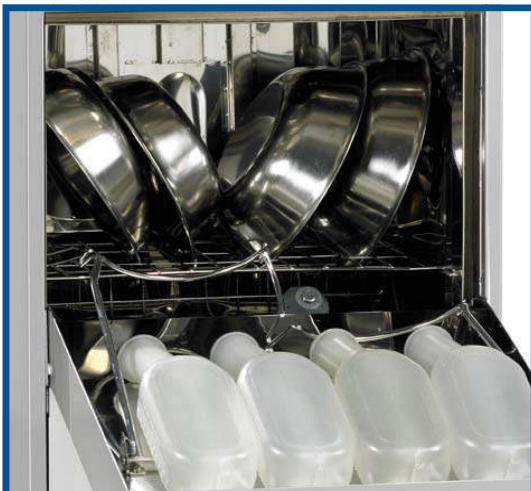

BOWLS

Do **not** empty foaming liquids into machine.

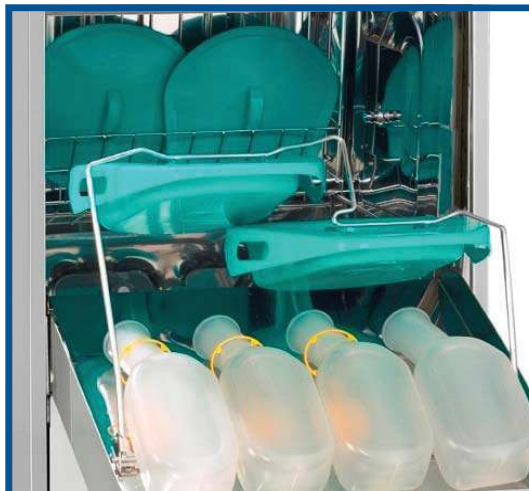

SLIPPER PANS, BOTTLES & LIDS

No pre-slucing required



BEDPANS AND BOTTLES

No pre-slucing required



LARGE BOWLS

Maximum 2 per load

CYCLE INFORMATION - TOUCH SCREEN DISPLAY



ABOVE: TOUCH SCREEN USER DISPLAY PANEL

NORMAL: 10 MINUTES INCLUDING FLUSH, SHORT WASH & **DISINFECTION** IDEAL FOR BOWLS, BOTTLES.

INTENSIVE: 11 MINUTES INCLUDING FLUSH, WASH & **DISINFECTION** IDEAL FOR PANS AND BOTTLES.

EMPTYING: **NON DISINFECTION CYCLE** 1 MINUTE
(Note: This cycle may be disabled on your machine) FOR FLUSHING OUT CONTENTS OF PAN IF YOU DO NOT HAVE A FULL LOAD TO PROCESS.
CONFIRMATION OF NON-DISINFECTION CYCLE REQUIRED BY USER PRIOR TO START.

LONG: 13 MINUTES FLUSH, WASH, DISINFECTION. LONG EST PRE SET CYCLE WITH 5 MINUTE WASH PHASE. Useful for SOILED MOP BUCKETS, CYTOTOXIC WASTE.

IMPORTANT:

- DO NOT PLACE ANY ITEMS IN THE MACHINE THAT CANNOT GO INTO A TOILET. **NO WIPES, PAPER TOWEL, TESTING STRIPS, ETC.**
- **SMALL ITEMS** MUST BE PLACED IN THE UTENSIL BASKET PROVIDED.
- IF YOU ARE NOT SURE ASK A NURSE EDUCATOR



**SMALL ITEMS
MUST GO IN THE
UTENSIL BASKET
PROVIDED**



PLEASE CALL RHIMA AUSTRALIA ON 1300 347 944 FOR DETERGENTS, USER OR TECHNICAL ASSISTANCE