

# DEKO 190GT WASHER DISINFECTOR - LOADING INSTRUCTIONS

(version: 08/20)



**BOWLS**  
Do **not** empty foaming liquids into machine.



**SLIPPER PANS, BOTTLES & LIDS**  
**No** pre-slucing required



**BEDPANS AND BOTTLES**  
**No** pre-slucing required



**LARGE BOWLS**  
Maximum 2 per load

## CYCLE INFORMATION - TOUCH SCREEN DISPLAY



ABOVE: TOUCH SCREEN USER DISPLAY PANEL

**NORMAL:** 10 MINUTES INCLUDING FLUSH, SHORT WASH & **DISINFECTION** IDEAL FOR BOWLS, BOTTLES.

**INTENSIVE:** 11 MINUTES INCLUDING FLUSH, WASH & **DISINFECTION** IDEAL FOR PANS AND BOTTLES.

**EMPTYING:** **NON DISINFECTION CYCLE** 1 MINUTE  
(Note: This cycle may be disabled on your machine) FOR FLUSHING OUT CONTENTS OF PAN IF YOU DO NOT HAVE A FULL LOAD TO PROCESS.  
CONFIRMATION OF NON-DISINFECTION CYCLE REQUIRED BY USER PRIOR TO START.

**LONG:** 13 MINUTES FLUSH, WASH, DISINFECTION. LONG EST PRE SET CYCLE WITH 5 MINUTE WASH PHASE. Useful for SOILED MOP BUCKETS, CYTOTOXIC WASTE.

### IMPORTANT:

- DO NOT PLACE ANY ITEMS IN THE MACHINE THAT CANNOT GO INTO A TOILET. **NO WIPES, PAPER TOWEL, TESTING STRIPS, ETC.**
- **SMALL ITEMS** MUST BE PLACED IN THE UTENSIL BASKET PROVIDED.
- IF YOU ARE NOT SURE ASK A NURSE EDUCATOR



**SMALL ITEMS  
MUST GO IN THE  
UTENSIL BASKET  
PROVIDED**



PLEASE CALL RHIMA AUSTRALIA ON 1300 347 944 FOR DETERGENTS, USER OR TECHNICAL ASSISTANCE