DEKO 190IX WASHER DISINFECTOR - LOADING INSTRUCTIONS





BOWIS

Do notempty foaming liquids into machine.



SLIPPER PANS, BO TILES & LIDS No pre-sluic ing require d



BEDPANS AND BOTHES

No pre-sluic ing require d



LARGEBOWIS
Maximum 2 perload

CYCLE INFORMATION - TO UCH SCREEN DISPLAY



ABOVE: TO UCH SCREEN USER DISPLAY PANEL

NORMAL: 10 MINUTES INCLUDING FLUSH, SHORT WASH & DISINFECTION IDEAL FOR BOWLS, BOTTLES.

INTENSIVE: 11 MINUTES INCLUDING FLUSH, WASH & DISINFECTION IDEAL FOR PANS AND BOTILES.

EMPIYING: NON DISINFECTION CYCLE 1 MINUTE

(Note: This cycle may be disabled on your machine) FOR FLUSHING OUT CONTENTS OF PAN IF YOU DO NOT HAVE A FULLIOAD TO PROCESS.

CONFIRMATION OF NON-DISINFECTION CYCLE

IONG: 13 MINUIES FLUSH, WASH, DISINFECTION. LONG EST PRE SET CYCLE WITH 5 MINUIE WASH PHASE Use ful for SOILED MOP BUCKETS, CYTOTOXIC

REQUIRED BY USER PRIOR TO START.

WASTE

IMPORTANT:

- DO NOT PIACE ANY ITEMS IN THE MACHINE THAT CANNOT GO INTO A TOILET NO WIPES, PAPER TO WEL, TESTING STRIPS, ETC.
- SMAIL FIEMS MUST BE PLACED IN THE UTENSIL BASKET PROVIDED.
- IF YO U ARE NO TSURE ASK A NURSE EDUCATOR.



SMALL FIEMS
MUSTGO IN THE
UTENSIL BASKET
PROVIDED

0

PLEASE CALL RHIMA AUSTRALIA ON 1300 347 944 FOR DETERGENTS, USER OR TECHNICAL ASSISTANCE