

# RHIMA DVS and DVS+ WASHER DISINFECTORS LOADING INSTRUCTIONS



## BOTTLES

Do **not** empty foaming liquids into machine.



## BOTTLES & PANS

No pre-sludging required.



## BOTTLES, BEDPANS AND LIDS


No pre-sludging required.



## BOWLS AND BUCKETS

Maximum 1 per load.

## CYCLE INFORMATION - TOUCH SCREEN DISPLAY

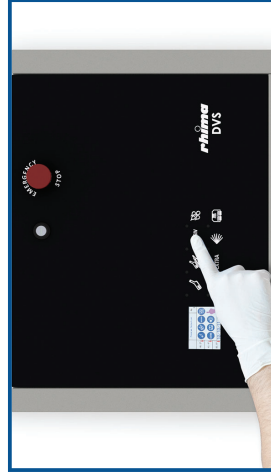

DVS TOUCH SCREEN USER DISPLAY PANEL

**URINALS:** **7 minutes** includes cold rinse, hot washes with detergent, disinfection, hot rinse then cooling, ideal for bottles.

**NORMAL:** **10 minutes** includes cold rinses, hot washes with detergent, disinfection, hot rinse then cooling, ideal for pans and bottles.

**INTENSIVE:** **12 minutes** includes cold rinses, hot washes with detergent, disinfection, hot rinse then cooling, ideal for pans, bottles and bowls.



**MULTIWASHER:** **12 minutes** includes cold rinses, hot washes with detergent, disinfection, hot rinse then cooling, ideal for utensils and small items (Using the optional basket).



In addition to the infrared sensor on the DVS+, the pedal control switch can be used to open and close the wash chamber door.

**DVS+ INFRARED SENSOR CONTROL OPERATION**  
(Place hand in front of sensor for 3 seconds)

## IMPORTANT:

- Do not place any items in the machine that cannot go into a toilet. NO WIPES, PAPER TOWEL, TESTING STRIPS, ETC.
- Small items must be placed in the utensil basket provided on request.
- If you are not sure ask a Nurse Educator



**SMALL ITEMS  
MUST GO IN  
THE OPTIONAL  
UTENSIL BASKET**